


MDHearingAid *FIT* Diagram

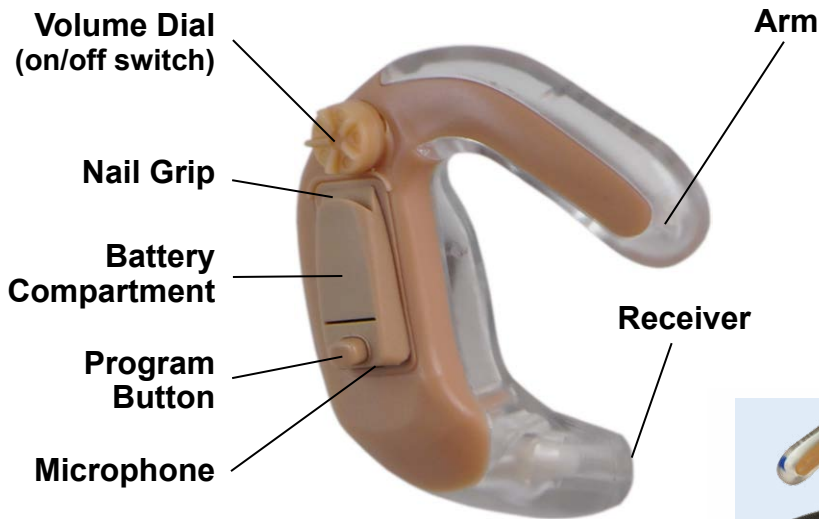

Packaged with (optional)
Comply™ Premium Foam Tip attached

Four Easy Steps for using Your MDHearingAid *FIT*



Step 1: Insert Battery

a) Remove the yellow tab from battery and let stand for one (1) minute to activate. Next, verify the Hearing Aid is turned OFF by turning the Volume Dial away from the Arm until you feel a click or until Volume Dial no longer turns.

b) Pull the Nail Grip downward to open the Battery Compartment. With the red + side of Battery Compartment facing up, insert the Battery.

c) Push the Battery Compartment closed.

Please note: If Battery Compartment does not close easily, battery is inserted upside down.

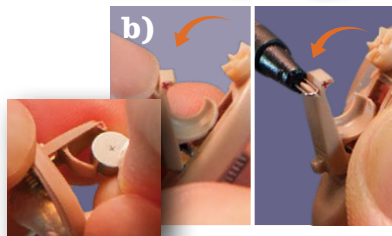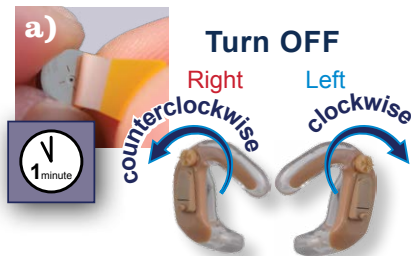


Step 2: Using Optional Comply™ Premium Foam Tips

Some *FIT* users find using a Tip provides improved comfort and performance. **The Tip comes attached** so you can give it a try.

a) Push the Tip until you feel the end of the Receiver through the opening of the Tip.

b) Gently pinch the Tip between your index finger and thumb to compress the foam.


c) Place compressed Tip in ear canal entrance (NOT inside of ear canal), with the volume dial facing outward, and proceed to Step 3b.

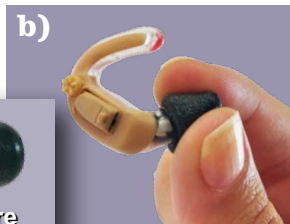


Step 3: Place MDHearingAid *FIT* Into Your Ear

a) If not using a Tip, place Receiver in the entrance to your ear canal NOT inside your ear canal.

b) Tuck Arm into the upper bowl (concha) of your ear.

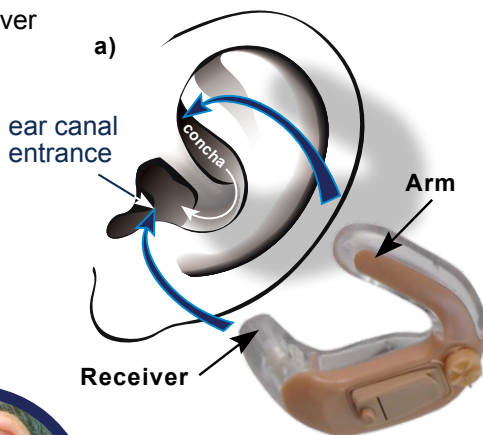

MDHearingAid *FIT*
in final position

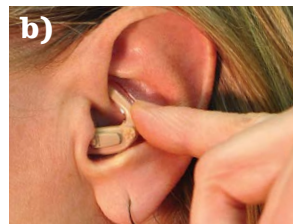


Step 4: Turn the MDHearingAid *FIT* On

The MDHearingAid *FIT* uses a Volume Dial with a built in on/off switch (like a radio).

- a) To turn hearing aid ON, rotate Volume Dial towards the Arm, past the click.
- b) To turn hearing aid OFF, rotate Volume Dial away from the Arm, past the click or until Volume Dial no longer turns.

Turn ON

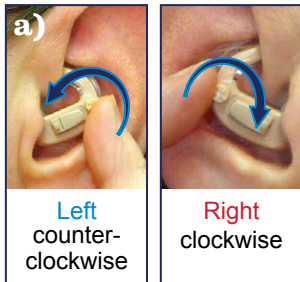

IMPORTANT: When the Hearing Aid is turned ON, it will take a few seconds to begin amplifying sound. Please raise the volume SLOWLY to avoid a sudden blast of sound.

Alternative On/Off Control: Once the desired volume is reached, you may open and close the Battery Compartment door to turn aid ON and OFF.

Turn OFF

