


MDHearingAid[®] >>>PRO

THE DOCTOR'S CHOICE FOR AFFORDABLE HEARING AIDS[®]

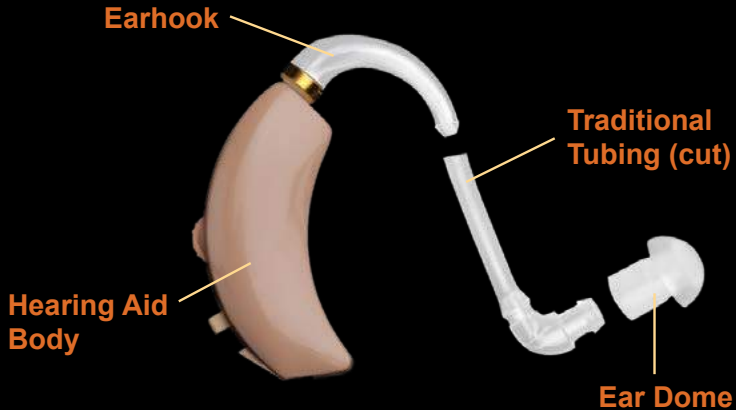

Traditional Tubing Manual

Step 1: Select the Correct Ear Dome Size


Test Ear Dome size by attaching an Ear Dome to the Tubing before inserting Ear Dome into ear canal.

Attach Ear Dome to Tubing

a) Hold the Elbow of the Tubing with one hand and hold the 'mushroom' top of the Ear Dome with the other hand.

b) Push the Ear Dome over the ridges on the end of the Elbow.

c) Pull lightly in the reverse direction to make sure the Ear Dome is attached securely.

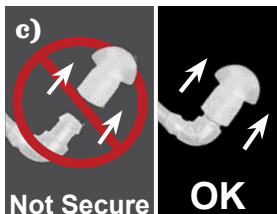

Step 2: Insert Ear Dome in Ear Canal

Hold the Elbow of the Tubing, with Ear Dome attached, and insert Ear Dome with gentle pressure into the ear canal. The 'mushroom' top of the Ear Dome should fit snug not tight.

Test Ear Dome for fit

Nod your head up and down and side to side while simulating chewing; if the Ear Dome slips out easily, it is too small.

Repeat with different sized Ear Domes to find the best fit for you.



Ear Canal



Step 3: Attach the Earhook and Cut Tubing Length

a) Attach the **PRO** Earhook to the threaded end of the Hearing Aid Body by gently pushing and twisting approximately 3 revolutions.

b) Place the Hearing Aid Body behind the ear so the Earhook rests comfortably on top of the ear (outside of any eyewear). Insert Ear Dome, with Tubing attached, into ear canal, and align Tubing next to Earhook.

c) Use a pen and mark a line on the Tubing ABOVE the bottom of the Earhook tip. Cut Tubing at marked line.

Please note: Tubing must be measured and cut separately for **each** of your ears.

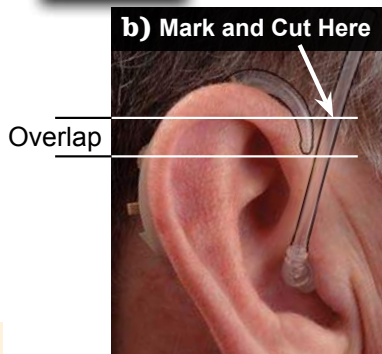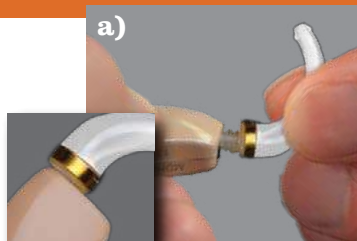



Measure twice, cut once!

Step 4: Attach Traditional Tubing to Earhook

a) Attach Tubing to Earhook by pushing the Tubing up onto the Earhook tip.

b) Twist the Tubing back and forth until the Tubing slides OVER the Earhook tip.

Make sure the 'mushroom' top portion of the Ear Dome points into the ear canal.

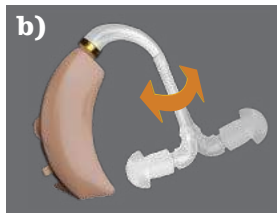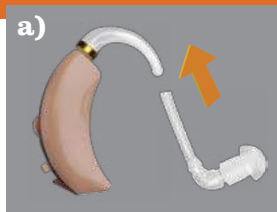


Step 5: Place Hearing Aid Behind Ear and Insert Ear Dome

a) Place the Hearing Aid Body over the top of the ear so the curve of the Earhook rests comfortably (outside of any eyewear).

b) Insert Ear Dome with gentle pressure into the ear canal until the Ear Dome fits securely.

IMPORTANT: If the Ear Dome is not positioned securely in your ear canal, the Hearing Aid may whistle when it is turned on. This is normal. **Once the Ear Dome is properly inserted in your ear canal, the whistling should stop.** (See User Manual pages 24-25 for more Troubleshooting tips.)

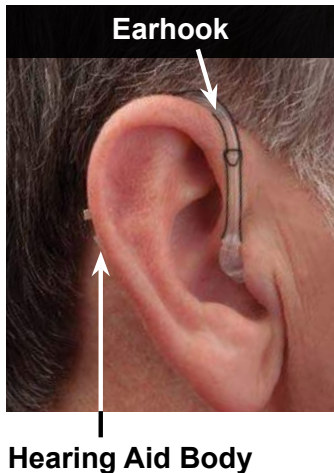



www.MDHearingAid.com

Printed in U.S.A.

02.15